| GRIFFINE ENDUCTION  01/02/2017                            | CLEANING and DESINFECTION AGENTS |  |        |          |        |        |          |       |       |                                       |                     |          |         |        |        |       |        |         |                |
|---|----------------------------------|--|--------|----------|--------|--------|----------|-------|-------|---------------------------------------|---------------------|----------|---------|--------|--------|-------|--------|---------|----------------|
| color signification : No damage partially damaged damages | desinfect effect                 | chemical products  | BOEING | COMBI    | COMETE | COSMOS | DIABOLO  | EBENE | ABAKA | Gamme Esprit<br>(dotty, dream , cosy) | Gamme GINKGO (wave) | MEDISTAT | MUNDIAL | SELECT | SIRIUS | URBAN | ADDICT | NEPTUNE | ELFORM/DENFORM |
| Dürr  | Х                                | FD 366<br>FD 333<br>FD 360   | 0      | 0        | 0      | 0      | 0        | 0     | 0     | 0                                     | 0                   | 0        | 0       | 000    |        | 0     | 0      | 0       | 0              |
| ANIOS   | X                                | Aniosurf Premium Surfanios Premium D.D.S.H. Surfa'Safe Surfa'Safe premium  |        |          | 0      | 0      | 0        | 0     |       |                                       |                     | 0        |         |        |        |       |        | 0       | 0000           |
| Softcare  |                                  | Softcare cleaning  |        |          |        |        |          |       |       |                                       |                     |          |         |        |        |       |        |         |                |
| Lederzentrum  |                                  | Lederzentrum products  |        |          |        |        |          |       |       |                                       |                     |          |         |        |        |       |        |         |                |
| Diversey<br>Mc Donald                                     |                                  | Teepol diluted at 1% in water<br>désinfectant DR<br>wash detergent HA<br>lobby SK care<br>sanitizer  |        |          | 0      | 0      | 0 0 0    | 0 0 0 |       |                                       | 0                   | 0 0 0 0  |         |        | 00000  | 00000 | 00000  | 0000    | 0000           |
| Henkel  | Х                                | Incidin foam diluted 50% in water Incidin Extra N  |        | 0        |        | 0      | 0        |       | 0     |                                       |                     | 0        | 0       |        |        |       | 0      |         |                |
| MERZ  |                                  | Pursept A  |        |          |        | 9      | 9        | 2     | -     |                                       |                     | 2        | 9       | 9      | 9      | 9     | 9      |         |                |
| ALPRO   | X                                | Plastisept   |        |          |        | 2      |          | -     | -     |                                       |                     | -        | 9       | 9      | 9      | 9     | 9      |         |                |
| BECHT<br>Schülke  |                                  | Bechtozid Premium<br>Mikrozid AF swipe<br>Mikrozid sensitive liquide<br>Terralin diluted 50% in water (eau)                                | •      | •        | •      | •      | •        | •     | •     |                                       | •                   | •        | •       | •      |        |       |        |         | •              |
| COSIMEDE  |                                  | Stolfenberg  |        |          |        |        |          |       |       |                                       |                     |          |         |        |        |       |        |         |                |
| HARMANN/BODE  | Х                                | Schnell-Desinfektion Desinfektionsschaum Bacillol plus pure Bacillol plus diluted 20% in water Bacillocid rasant diluted 0,5% in water     |        |          |        |        |          |       |       |                                       |                     |          |         |        |        |       |        |         |                |
| IONTO   | Х                                | Care clean   |        |          |        |        |          |       |       |                                       |                     |          |         |        |        |       |        |         |                |
| eCOLAB  |                                  | Actichlor Klercide CRBiocides E, A, B, D Kerclean CRSterile Detergent Klercide CRBiocides X, C, Y Klercide 70/30                           |        | 0        | 0      | 0      | 0        | 0     | 0000  |                                       |                     | 0        |         |        |        |       |        | 0       | 0000           |
| Alpro   | Х                                | PlastiSept<br>MinutenSpray classic   | 0      | <b>O</b> | 0      | 0      | <b>O</b> | 0     | 0     | 0                                     | 0                   | 0        | 0       | 0      | 0      | 0     | 0      | 0       | 0              |
| Metrex  |                                  | CaviCide   |        |          |        |        |          |       |       |                                       |                     |          |         |        |        |       |        |         |                |
| Henry Schein  | Х                                | Maxima Disinfection  |        |          |        |        |          |       |       |                                       |                     |          |         |        |        |       |        |         |                |
| GENERAL   |                                  | eau savonneuse // soapy water Isopropanol alcohol /water (70%/30%) Eau oxygénée (type médical) // hydrogen peroxyde (healthcare reference) | •      | •        | •      | •      | •        | •     | •     | •                                     | •                   | •        | •       | •      | •      | •     | •      | •       | •              |
|   |                                  | Hydroxyde ammonium diluted in water (concentration 5%)   | •      |          | •      | •      |          | •     | •     |                                       | •                   | •        |         |        |        |       |        | •       | •              |
|   | }                                | Autoglym vinyl-rubber and leather or GS27  Acid acetic diluted in water (concentration 2%)   | •      |          | •      |        |          |       |       |                                       | •                   |          |         |        |        |       |        |         |                |

## **Recommendations:**

All cleaning agents have been evaluated at the date indicate on the top. GRIFFINE is not responsible if producers of substances modify something between our evaluation and actual date that generate damage and not correspond to our test.

It is necessary to respect instructions from these cleaning agents (dilution, ...). BUT after cleaning, we recommend to rinse the surface with water. Indeed some cleaner contain additives based on chemical salts like quaternary ammonium for example. They can generate stains on surface in the time or damages after drying because you don't rinse with water. GRIFFINE can not be responsible of this.

These tests have been realized on our products before any transformation. After some technical steps and transformation (heating steps or others...), these properties evaluated can be different, Also it can not be the responsability of GRIFFINE. A previous test is necessary to confirm if our product react exactly with the same behavior and no damages appeared